


## Problem: Standard User Privilege

### For Windows 7

**Problem:** Standard User Privilege, Run CardioScan12 or Resting ECG, etc., shows the below:

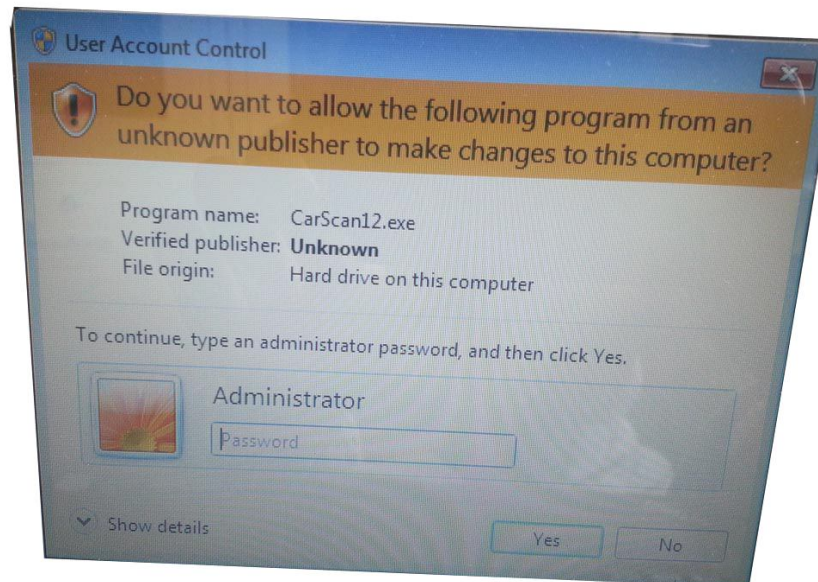

### How to resolve this issue?


1. Download "**Microsoft Application Compatibility Toolkit**"

<http://download.microsoft.com/download/D/3/3/D33A8F5D-A7E1-4BBE-B04D7A688785A229/ApplicationCompatibilityToolkitSetup.exe>


2. Administrator log on, to install ApplicationCompatibilityToolkitSetup.exe

3. Start-All Program - Microsoft Application Compatibility Toolket-Compatibility Administrator (32-bit), Custom Database - Create New - Application Fix...

## Problem: Standard User Privilege

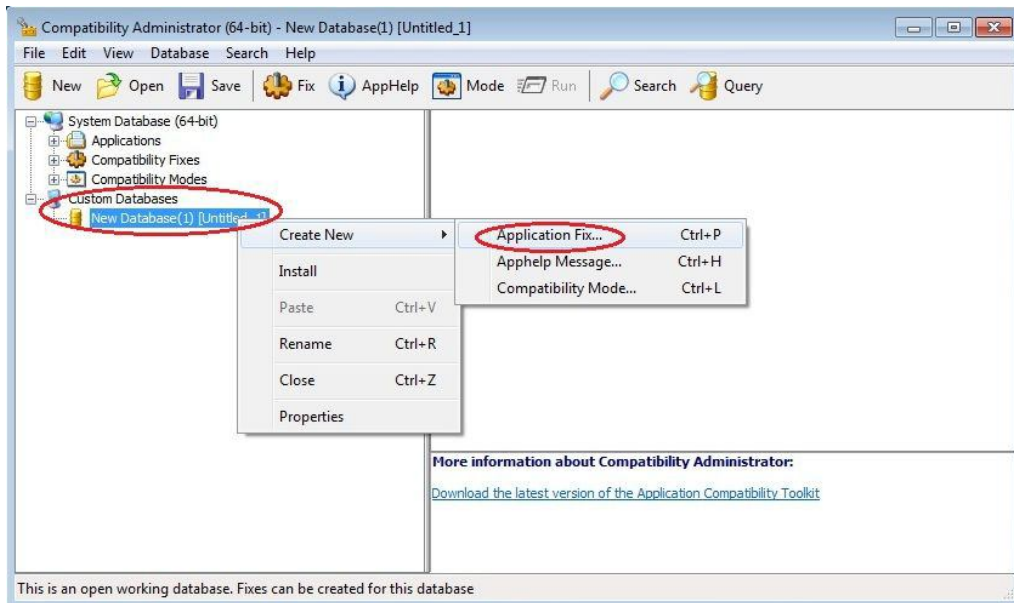

Add the white list application...Next...

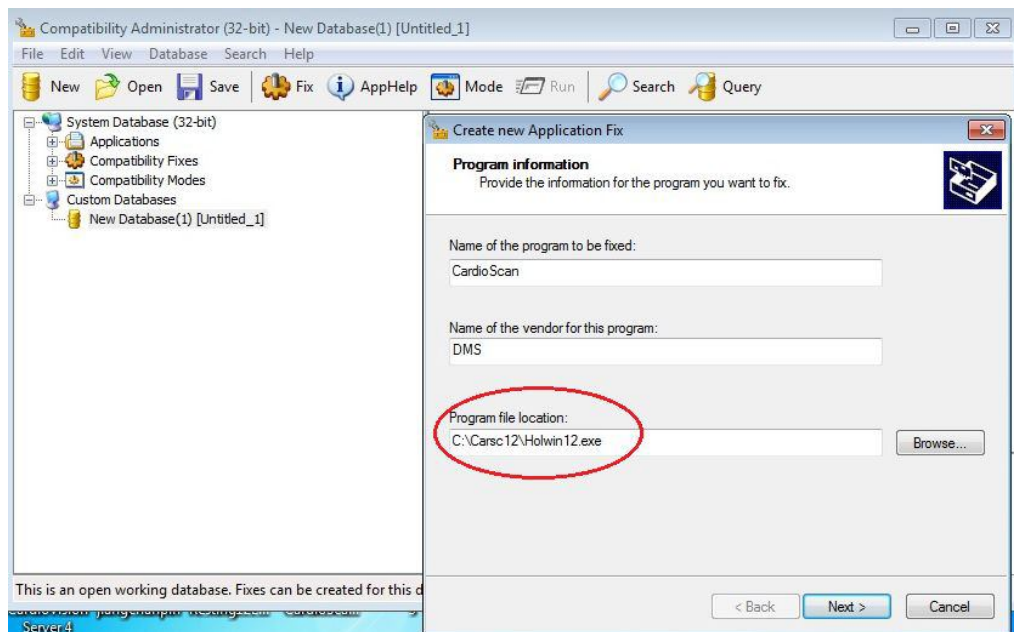


**Note:**


If C:\Carsc12\Holwin12.exe file doesn't exist, please choose C:\Carsc12\CarScan12.exe.

## Problem: Standard User Privilege

Choose “RunAsInvoker”...Next...

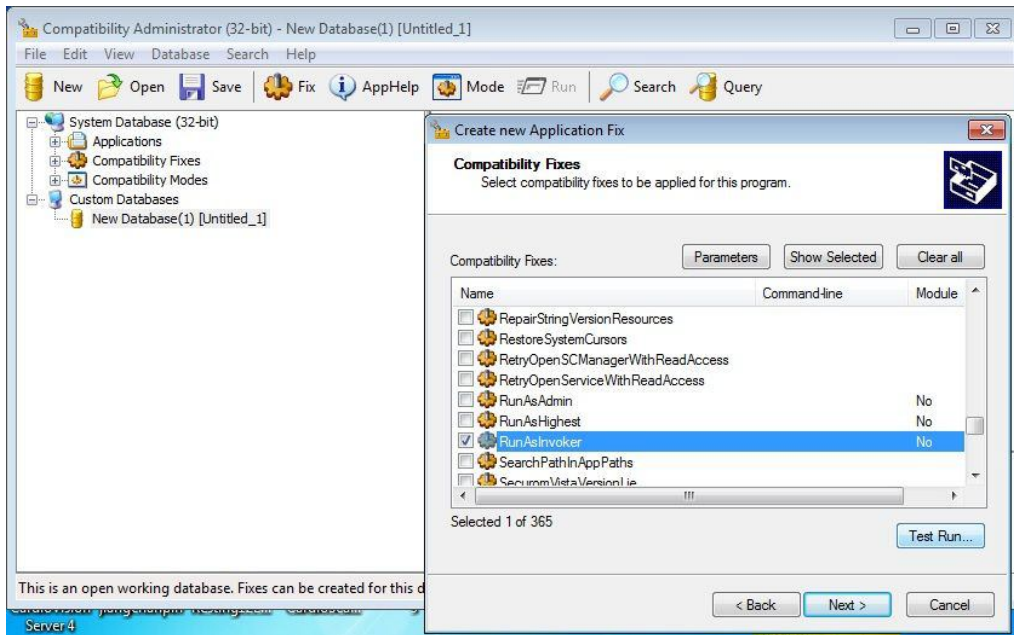

Click **Save** to get .sdb file

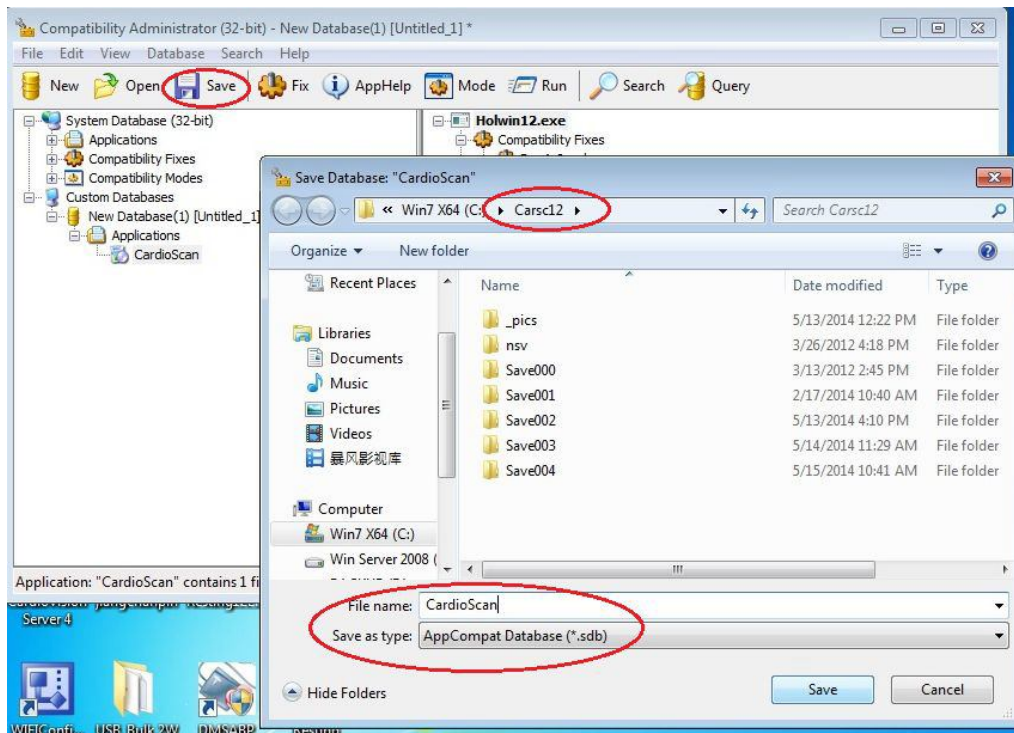

## Problem: Standard User Privilege

Click **File - Install...** Shows "The database was successfully installed". **ALL DONE.**

